



## - *Compassionate Colposcopy*<sup>®</sup>

### **Endocervical and Ectocervical Histology Brushes: Bristle and Fabric-Based Devices:**

#### **Inspect the brushes to assure an adequate sample is obtained.**

Inspect the bristles or hook arrays for trapped tissue prior to placement in the fixative vial. For endocervical fabric-based devices the hook array should fill completely when used according to package insert guidance and proper fit or pressure on the endocervical mucosa during rotation. For ectocervical frictional biopsy, with proper pressure and sufficient rotation, the device should fill within two applications (sites) on the cervix.

**Inspect the device tip for sufficient trapped tissue prior to placement in the vial.**

**Ectocervix Goal: Brush tip should be at least half full of tissue**



**SoftBiopsy<sup>®</sup>**



**Spirabrush CX<sup>®</sup> tip**

**Endocervix Goal: Brush tip should be nearly filled with tissue**

